ENGLISH

Wrist Blood Pressure Monitor

Instructions for use 3

ESPAÑOL

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Manual de instrucciones 34

FRANÇAIS

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Mode d'emploi 67

Manufactured for/Manufacturado para/
 Fabriqué pour :
 Beurer North America LP
 900 N Federal Highway, Suite 300
 Hallandale Beach, FL 33009
www.beurer.com

Questions or comments?

Call toll free 1-800-536-0366 or contact info@beurer.com

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READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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ENGLISH

CONTENT

1. IMPORTANT SAFETY INSTRUCTIONS	4
2. Introduction	14
3. Important information.....	14
4. Package Contents	15
5. Parts and Controls.....	16
6. Preparing for Measurement	17
7. Measuring Blood Pressure.....	19
8. Evaluating Results.....	22
9. Classification Categories	22
10. Displaying and deleting measurements	23
11. Care and Maintenance	27
12. Troubleshooting Guide	28
13. Technical Specifications	28
14. FCC Statement	30
15. Warranty.....	31

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards, Your Beurer Team

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

WARNING

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

NOTICE

NOTICE addresses practices not related to personal injury, such as product and/or property damage.

WARNING:

- Consult your physician or pharmacist before starting to measure your blood pressure.
- Read and understand all instructions and warnings before using this device.
- Like any oscillometric blood pressure measurement devices, certain medical conditions can affect the measurement accuracy, among others:
 - disorder of the cardiac rhythm
 - very low blood pressure
 - patients in shock
 - patients with very low blood perfusion
 - diabetes
 - vessel anomalies
 - people with electrical implants, such as a cardiac pacemaker
 - women who are pregnant.
- Due to their condition the oscillometric measurement method can produce incorrect readings. This represents a risk for your health, since values may be interpreted incorrectly. Always consult your physician to determine what will be suitable for you.
- This product does not and is not intended to provide a medical diagnosis. Measurements results are for reference only. Self-diagnosis and treatment, e.g. regarding medication, using measured results represent a risk for your health. Always consult with a licensed physician for determination of appropriate medication and dosage thereof. Follow the instructions of your physician or licensed healthcare provider. If you have or suspect that you have a medical problem, promptly consult your physician. If you have an emergency please call 911 immediately.
- Only a physician or a trained health care professional who is familiar with your medical history is able to accurately interpret your blood pressure measurements. Consult your physician before starting blood pressure monitoring.

- This product is not intended to serve as a substitute for the advice of a physician or medical professional. This product is not intended to substitute for regular medical checkups. Contact your physician for specific information about your blood pressure.
- Please note that technically related measuring tolerances are possible. Please see section '13. Technical Specifications' for details.
NOTE: A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.
- Any cuff related blood pressure measurement in high repetition rates, can lead to severe measurement side effects, e.g.
 - any nerv compression with temporary arm/ hand paralysis
 - the release of an arterial or venous thrombus, which can cause a life threatening situation.
- Please contact your physician about the specific risks of cuff pressure in your specific case.
- The 'irregular heartbeat' function does not replace a cardiac examination, but may help to detect potential pulse irregularities at an early stage. Always consult your physician to determine what will be suitable for you.
- The 'irregular heartbeat' function is not designed for diagnosing or treating an arrhythmic disorder. Arrhythmia can only be ascertained by a licensed physician.
- The WHO chart is not intended to replace a medical diagnosis. This chart is only a reference for different classifications of blood pressure.
- If you notice abnormal or suspicious variations in blood pressure measurements, consult your physician immediately.

- Women who underwent a breast or axillar lymph node removal operation should consult a physician or licensed healthcare provider before starting blood pressure measurements.
- Prior to use, you have to ensure that the blood pressure monitor is free of damage. If in doubt, do not use the unit and contact customer service at 1-800-536-0366.
- This device is intended only for personal, non-commercial, adult use in measuring blood pressure and pulse rate. Do not use the product for any other purpose. This product is not intended for use in a hospital, physician's office, or any other health care facility.
- The unit must be used in accordance to the specified ambient conditions, otherwise the accuracy of readings might be affected. See section '11. Care and Maintenance' and section '13. Technical Specifications' for details.
- Do not wrap the cuff around body parts other than your wrist. Misuse represents a risk to your health.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Please do not share the cuff with other persons, to avoid cross infection.
- Use this monitor only on humans.
- This product is not intended for use by or on children, toddlers and infants or on persons who cannot express their consent, e.g. persons with mental disorders or the like. Consult your physician for alternative methods of measuring a child's blood pressure.
- This product is not a toy. Keep it out of reach of children, toddlers, and infants.
- Keep the product out of the reach of pets.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
- This product contains small parts that may present a choking hazard to children. Keep the unit and all parts out of reach of chil-

dren. NEVER LEAVE CHILDREN OR THOSE WHO REQUIRE CLOSE SUPERVISION UNATTENDED WITH THIS DEVICE.

- People with disabilities, activity limitations, or who are physically frail, should be assisted by another person when using this unit.
- Proper cuff size is important for accurate measurements. Only use the device on adults who have the right wrist circumference for this instrument. See section '13. Technical Specifications' for suitable wrist circumferences.
- Do not use any cuffs, parts and accessories other than those explicitly recommended by the manufacturer for use with this product. Parts and accessories not approved for use with the device may cause damage to your health and to the product.
- Electromagnetic interference: Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens) while it is in operation, as inaccurate measurements may result.

To prevent such interference, use the unit at a sufficient distance from such devices or turn the devices off.

- Do not measure your blood pressure while operating a vehicle or in any situation which requires your full attention.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the wrist.
- Avoid any mechanical restriction, compression or bending of the cuff.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
- Consult your physician if you have any doubt using the blood pressure monitor simultane-

ously with other monitoring medical equipments on the same limb.

- Do not use the cuff on people who have undergone a mastectomy.

CAUTION

- The device should not be used when your arm has been wounded/injured or when a catheter has been inserted. Such use may result in injury.
- Remove any kind of wrist jewelry or the like before taking a measurement. This could cause bruises.
- Do not place the cuff over heavy clothing (e.g. a jacket or sweater sleeve) as the blood pressure monitor will not be able to take a proper measurement and there is an elevated danger of acquiring hematoma or skin marks during the course of the measurement.
- In case the cuff does not stop inflating, interrupt the measurement by pressing the start/stop button and open the cuff at once.
- Do not disassemble the unit, it may result in injuries.
- When applying the cuff, make sure there are no wrinkles in the cuff. This could cause bruises.
- Blood pressure measurements can lead to temporary marks on the skin at the site of the cuff placement. This is especially the case in high repetition rates, in hypertonic patients and in patients with weak pulses. In rare cases a mark may persist for a couple of days. Please contact your physician about these specific risks of cuff pressure in your specific case.
- For hygienic reasons the cuff is intended only for the use by one person.
- It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.
- Prolonged over-inflation (cuff pressure exceeds 300 mmHg or maintained above

- 15mmHg for longer than 3 minutes) of the bladder may cause ecchymoma of your arm.
- Please note that when inflating, the functions of the limb in question may be impaired.
 - Do not place the cuff over wounds as this may cause further injury.
 - The accuracy of any blood pressure measurement with this device can be affected by a multitude of causes. Some can be avoided some have to be accepted at the time of measurement. Speak with your physician about possible causes affecting your blood pressure.
 - Do not inflate the cuff unless wrapped around wrist.
 - Do not touch the unit and cuff and/or press any buttons when measurement is in progress, except for stopping the measurement. Doing so may cause incorrect readings.

NOTICE

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling. Protect the unit against hard knocks (e.g. dropping the unit), moisture, water, dirt, dust, chemicals, extreme hot or cold temperatures, major temperature fluctuations, direct exposure to sunlight and heat sources which are too close (e.g. stoves, heating radiators). This may damage the unit.
- The device must be stored in the specified ambient conditions. Please see section '11. Care and Maintenance' and section '13. Technical Specifications' for details.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Never immerse and or spill water or any other liquid onto the monitor or any components,

otherwise liquid will enter it and cause damage.

- Do not wash the unit in a washing machine, dish washer or tumble dryer.
- Observe the local regulations for material disposal. Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- Never attempt to repair, open and/or disassemble the unit (including wrist cuff and optional accessory) or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.
- Do not drop or insert any object into any opening. This may damage the unit.
- Do not press the buttons with excessive force or with pointed objects.
- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for 3 minutes.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- If you are using the unit for the first time, please remove the protective film from the display.
- The monitor can maintain the safety and performance characteristics for a minimum of 10.000 measurements or 3 years.
- Further documents which include information for the Calibration of the device can be asked at the service address.
- The device can be tested in case of supposed inaccuracy every two years. Please contact the nearest service address.

Cleaning and storing the device and cuff

To keep your digital blood pressure monitor in the best condition and protect the unit from damage follow the directions listed below:

- Do not use any cleaning agents or solvents.
- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries.
- Clean your device and cuff carefully only with a slightly moistened soft cloth and dry it immediately with a soft dry cloth.
- When storing the device, make sure that no heavy objects are placed on top of it.
- Do not fold the cuff tightly. If the unit is stored near freezing, allow it to acclimate to room temperature before use.
- Always store the unit in its storage case after use.

- This unit may not meet its performance specifications if stored or used outside of the temperature and humidity ranges (see section '13. Technical Specifications').
- Use the unit consistent with the instruction provided in this manual.

Advice on use

- For consistency, always measure your blood pressure at the same time each day.
- Always wait at least 5 minutes between measurements to allow the blood circulation in your arm to return to normal. You may need to increase the wait time depending on your individual physiological characteristics. Because of normal physiologic changes and the many external factors influencing blood pressure, it is unusual to obtain identical blood pressure measurements, even when taken a few minutes apart. Speak with your physician to determine what is affecting your blood pressure and which variation may be seen as normal in your case.

- When performing multiple measurements on an individual, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising. You should not be physically tired or exhausted while taking measurement.
- Repeat the measurement if you are unsure of the results.
- Measurements are for your information only – they are no substitute for a medical examination. Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).
- Do not use the blood pressure monitor on newborns, pregnant women or patients with pre-eclampsia. Consult your doctor before using the blood pressure monitor during pregnancy.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- For reliable monitoring and reference of blood pressure, keeping long-term records is recommended.
- To minimize measurement variations due to physical activity, rest at least 5 minutes before measuring your blood pressure (or 15 minutes after strenuous activity).
- Stress raises blood pressure.
- Perform measurements in a quiet and relaxed environment at room temperature.

Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or recharge-

able batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.

- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions).
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

2. Introduction

This blood pressure monitor performs non-invasive measurement and monitoring of adult arterial blood pressure. You can use it to measure your blood pressure quickly and easily, store the results, and display stored readings and their average. This device also features an irregular heartbeat indicator, with values classified and graphically evaluated according to WHO guidelines.

3. Important information

Signs and symbols


The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Attention
	Note Note on important information
	Observe the instructions for use
	Application part, type BF (cuff)
	Direct current
Storage 	Permissible storage temperature and humidity
Operating 	Permissible operating temperature and humidity
	Protect from moisture
SN	Serial number

4. Package Contents


- 1 x Blood pressure monitor
- 1 x Wrist cuff for wrist circumferences
5.5 in–7.7 in (14–19.5 cm)
- 1 x Storage case
- 2 x 1.5V AAA batteries
- 1 x Instruction manual
- 1 x Quick start guide

5. Parts and Controls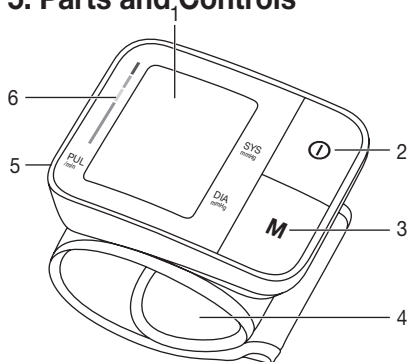

1. LCD Display
2. Start/Stop Button ①
3. Memory Button **M**
4. Wrist Cuff
5. Battery Compartment (on rear)
6. WHO indicator

LCD Display Information:

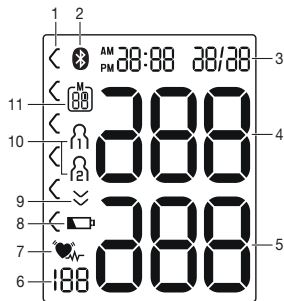1. WHO indicator
2. *Bluetooth*® Indicator
3. Time and Date
4. Systolic Pressure
5. Diastolic Pressure
6. Pulse Rate
7. Irregular Heartbeat Indicator /Pulse Rate Icon
8. Low Battery Indicator

9. Deflation Indicator
10. User Number Indicators  / 
11. Memory Space Number/Memory Display of Average (R), Morning (R_M), or Evening (P_M)

System requirements for the app:

Bluetooth® ≥ 4.0, iOS from version 8.0, Android™ from version 4.4.

List of supported devices:

6. Preparing for Measurement

Inserting/changing batteries:

1. Remove the Battery Compartment Cover from the rear of the device.

2. Insert two 'AAA' alkaline batteries (included) according to the polarity markings inside the compartment.

NOTE: Do not use rechargeable batteries.


3. Reattach the Battery Compartment Cover.
4. Before first use, remove the clear plastic film from the LCD Display.

If the battery change symbol  is flashing and  appears, no further measurements are possible and you must replace all batteries.

All LCD Display elements will briefly appear and '12h' will flash. Set the date and time as described below. Any saved measurements will be retained.

Setting the hour format, date, and time:

This menu allows you to set the following functions, one after another.

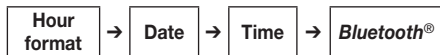If date and time are not set, stored measurements will not have a valid timestamp associated with them.

NOTE: Press and hold the M1 button to advance values rapidly.

Hour format

The hour format now flashes on the display.

- Select the desired hour format using the memory button **M** and confirm with the Start/Stop button **ⓘ**.

Date

The year flashes on the display.

- Select the desired year using the memory button **M** and confirm with the Start/Stop button **ⓘ**.

The month flashes on the display.

- Select the desired month using the memory button **M** and confirm with the Start/Stop button **ⓘ**.

The day flashes on the display.

- Select the desired day using the memory button **M** and confirm with the Start/Stop button **ⓘ**.

- ⓘ** If the hour format is set as 12h, the day/month display sequence is reversed.

The hours flash on the display.

- Select the desired hour using the memory button **M** and confirm with the Start/Stop button **I**.

The minutes flash on the display.

- Select the desired minute using the memory button **M** and confirm with the Start/Stop button **I**.

The *Bluetooth*® symbol flashes on the display.

- Use the memory button **M** to select whether automatic *Bluetooth*® data transfer is activated (*Bluetooth*® symbol flashes) or deactivated (*Bluetooth*® symbol is not shown) and confirm with the Start/Stop button **I**.


- i** *Bluetooth*® transfers will reduce the battery life.

7. Measuring Blood Pressure

The blood pressure monitor needs to be at room temperature when measuring. Measurements can be performed on the left or right arm.

Attaching the Cuff:

1. With your palm up, place the cuff around your left wrist about 0.4 in (1 cm) below the bottom of your hand.
2. Secure the cuff with the hook and loop strip so that it fits snugly but does not affect blood circulation.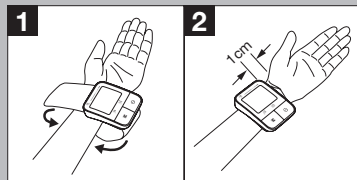


Correct posture

- Rest for at least five minutes before each measurement.

- Perform the measurement either sitting or lying down.

Always make sure that the cuff is level with your heart.

- To perform a measurement, sit comfortably with your arms and back supported. Do not cross your legs and place your feet flat on the floor.
- Keep still during the measurement and do not talk.
- Relax your arm and the palm of your hand

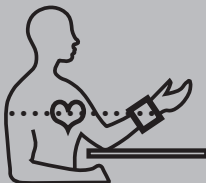Performing the blood pressure measurement

As described above, attach the cuff and adopt the posture in which you want to perform the measurement.

- Press the Start/Stop button ⓘ to start the blood pressure monitor. All displays will illuminate briefly.

The blood pressure monitor will begin the measurement automatically after 5 seconds.

The measurement is taken during the inflation phase.

- ⓘ You can cancel the measurement at any time by pressing the Start/Stop button ⓘ.

As soon as a pulse is found, the pulse symbol ♥ is displayed.

- The systolic pressure, diastolic pressure and pulse rate measurements are displayed.

- **EE_** appears if the measurement could not be performed properly. Observe the chapter on error messages/troubleshooting in these instructions for use and repeat the measurement.
- Now select the desired user memory by pressing the memory button **M**. If you do not select a user memory, the measurement is stored in the most recently used user memory. The relevant symbol **M1** or **M2** appears in the display.
- Press the Start/Stop Button **I** to switch off the blood pressure monitor. The measurement is then stored in the selected user memory.

Bluetooth® Data Transfer

If *Bluetooth®* data transfer is activated, data is transferred after pressing the Start/Stop button **I**.

- The *Bluetooth®* symbol flashes on the display. The blood pressure monitor now attempts to connect to the app for approx. 30 seconds.

- The *Bluetooth®* symbol stops flashing as soon as a connection is established. All measurement data is automatically transferred to the app. Once the data has been successfully transferred, the device switches off.
- If a connection to the app cannot be established after 30 seconds, the *Bluetooth®* symbol goes out and the blood pressure monitor switches off automatically after 3 minutes.
- If you forget to turn off the device, it will switch off automatically after a maximum of 3 minutes. In this case too, the value is stored in the selected or most recently used user memory.
- Wait at least 5 minutes before taking another measurement.

8. Evaluating Results

Irregular Heartbeat:

This device can detect irregular heartbeat patterns during measurement and will indicate them by displaying the irregular heartbeat indicator

This may be an indicator for arrhythmia, which is a condition where the heart rhythm is abnormal. These symptoms (omitted or premature heartbeats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress, or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the Irregular Heartbeat Indicator displays after a measurement. Rest for at least five minutes between measurements and

do not talk or move during measurement. If the icon  appears often, contact your doctor, since self-diagnosis and treatment based solely on the test results may be dangerous. It is vital to follow your doctor's instructions.

9. Classification Categories

Measurements can be classified and assessed by the table below. However, these standard values are only a general guideline, since individual blood pressure varies in different people and different age groups, etc.

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	160–179	100–109	seek medical attention

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	<120	<80	self-monitoring

Source: WHO, 1999

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the systolic and dia-

stolic values fall into two different categories, the classification Indicator on the unit always shows the higher category.

10. Displaying and deleting measurements

The results of every successful measurement are stored together with the date and time. The oldest measurement is overwritten in the event of more than 60 measurements.

- Once the device is switched off, select the desired user memory (1 2) using the memory button **M** and confirm with the Start/Stop Button **1**.

The average value of all measurements appears on the display.

User memory

If *Bluetooth®* is activated (the  symbol flashes on the display), the blood pressure monitor attempts to connect to the app. If you press the memory button **M** in the meantime, the transfer is cancelled and the average values are displayed. The  symbol is no longer shown. The buttons are deactivated and the  symbol ceases to flash as soon as a connection is established and the data is transferred.

Average values

R flashes on the display.

The average value of all saved measured values in this user memory is displayed.

- Press the memory button **M**.

AM flashes on the display.

The average value of the morning measurements for the last 7 days is displayed (morning: 5.00 a.m. – 9.00 a.m.).

- Press the memory button **M**.

PM flashes on the display.

The average value of the evening measurements for the last 7 days is displayed (evening: 6.00 p.m. – 8.00 p.m.).

Individual measured values

- When the memory button **M** is pressed again, the last individual measurement is displayed (in this example, measurement 03).
 - When the memory button **M** is pressed again, you can view your individual measured values.
 - To switch the device off again, press the Start/Stop Button **ⓘ**.
- i** You can exit the menu at any time by pressing the Start/Stop Button **ⓘ**.

Delete the measured values

- To clear the memory of the relevant user memory, you must first select a user memory.
- Start the retrieval of the average measured values. **A** flashes on the display and the average value of all saved measured values in this user memory is displayed.
- Press and hold the memory button **M** for 5 seconds depending on the user memory you are in.

All the values in the current user memory are deleted.

To clear individual measurements from the relevant user memory, you must first select a user memory.

- Start individual measurement access.
- Press and hold the memory button **M** for 5 seconds (depending on the user memory you are in).
- The selected value is deleted. The device briefly displays **CL 00**.
- If you would like to delete other values, repeat the process described above.

You can switch off the device at any time by pressing the **START/STOP** button **1**.

Transfer via *Bluetooth*®

It is also possible to transfer the measured values saved on the device to your smartphone via *Bluetooth*®.

You will need the 'beurer HealthCoach' app for this. The app is available from the Apple App Store and Google Play.

Proceed as follows to transfer values:

If *Bluetooth*® is activated in the settings menu, the data is transferred automatically after the measurement. The **Bluetooth** symbol appears in the top left of the display (see chapter 6 'Preparing for measurement').

Step 1: BC 57

Activate *Bluetooth*® on your device (see chapter 6 'Preparing for measurement, *Bluetooth*®').

Step 2: 'beurer HealthCoach' app

In the 'beurer HealthCoach' app, add the BC 57 under 'Settings/My devices'.

Step 3: BC 57

Take a measurement.

	
Step 4: BC 57: Data transfer immediately after measurements. • If <i>Bluetooth®</i> data transfer has been activated, data is transferred after having confirmed the user memory and pressing the START/STOP button ⓘ.	Step 4: BC 57: Data transfer at a later point: • Go to memory mode (chapter 10). Select the desired user memory. The <i>Bluetooth®</i> transfer starts automatically.

ⓘ The 'beurer HealthCoach' app must be active to allow data transfer.

If your smartphone has a protective cover, remove this to ensure that there is no interference during the transfer.

We would explicitly draw attention to the fact that the software to hand is not a medical product in accordance with EU Directive 93/42/EEC.

The values displayed are purely for visualisation purposes and must not be used as a basis for therapeutic treatments. The software is not part of a diagnostic-medical system.

11. Care and Maintenance

- It is recommended to disinfect the cuff 2 times every week if needed. Wipe the inner side (the side touching the skin) of the cuff with a soft cloth, squeezed after moistened with Ethyl alcohol (75-90%), then dry the cuff by airing.
- Cleaning of the cuff after the usage of every 200 times is recommended.
- Clean the device and cuff with a slightly moistened cloth.
- Do not use detergents or solvents.
- Never immerse the device in water or other liquids.
- When storing the device, make sure no heavy objects are placed on top of it.

- No component can be maintained by user in the monitor. Please contact the customer service in case of questions.
- The cuff integrity is maintained after 1,000 open–close cycles of the closure.

12. Troubleshooting Guide

- In case of faults, 'EE_' appears in the LCD Display, where '_' is a number from 0-8. Error messages may appear if:
- Systolic or diastolic pressure could not be measured (EE 1)
- Systolic or diastolic pressure was outside the measurement range ('H' or 'Lo' appear on the LCD Display)
- The cuff is fastened too tightly or loosely (EE2)
- Pressure is higher than 300 mmHg (EEB)
- Measurement time exceeds 160 seconds (EE3)
- Other system error (EE0, EE4, EE5, EE6)
- The data could not be sent via *Bluetooth*[®] (EE 7)

In such cases, repeat the measurement. Do not move or speak. If necessary, replace the batteries.

13. Technical Specifications

Model	BC 57
Type	KD-721
Measurement method	Oscillometric, non-invasive blood pressure measurement on the wrist
Measurement range	Cuff pressure 0-300 mmHg, Systolic 60-260 mmHg, Diastolic 40-199 mmHg, Pulse 40-180 beats/minute
Display accuracy	Systolic ± 3 mmHg, Diastolic ± 3 mmHg, Pulse ± 5 % of the value shown

Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: Systolic 8 mmHg / Diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	3 in (L) x 2.36 in (W) x 0.9 in (H) (78 mm (L) x 60 mm (W) x 23 mm (H))
Weight	Approximately 3.4 oz (96 g) (without batteries)
Cuff size	5.5 in–7.7 in (14–19.5 cm)
Permissible operating conditions	+50 °F to +104 °F (+10 °C to +40 °C), ≤85 % relative humidity (non-condensing)
Permissible storage conditions	-4 °F to 122 °F (-20 °C to +50 °C), ≤85 % relative humidity, 800-1050 hPa ambient pressure
Power supply	2 x 1.5V — AAA batteries

Battery life	For approx. 200 measurements, depending on levels of blood pressure and pump pressure
Accessories	Instructions for use, 2 x 1.5V AAA batteries, storage case
Data transfer via <i>Bluetooth</i> ® wireless technology	The blood pressure monitor uses <i>Bluetooth</i> ® low energy technology, 2.4 GHz frequency band, Compatible with <i>Bluetooth</i> ® ≥ 4.0 smartphones/tablets

The serial number can be found on the unit itself or in the battery compartment.

- This device is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Services

address or found at the end of the instructions for use.

- This device corresponds to the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktgesetz) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (noninvasive sphygmomanometers, Part3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2-30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

14. FCC Statement

Changes or modifications to the product not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and

can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct

the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

15. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Wrist Blood Pressure Monitor, Model BC 57, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Wrist Blood Pressure Monitor, Model BC 57, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase,

regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow lim-

itations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.

Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any

and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. For more information regarding our product line in the USA, please visit: www.beurer.com.

Manufactured for:
Beurer North America LP
900 N Federal Hwy, Suite 300
Hallandale Beach, FL 33009. USA
www.beurer.com

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Electromagnetic Compatibility Information

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration – electromagnetic emission

The BC 57 is intended for use in the electromagnetic environment specified below.
The customer or the user of the BC 57 should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 2	The BC 57 must emit electromagnetic energy in order to perform its intended function. Nearby electronic equipment may be affected. The BC 57 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration – electromagnetic immunity

The BC 57 is intended for use in the electromagnetic environment specified below. The customer or the user of the BC 57 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacture's declaration – electromagnetic immunity

The BC 57 is intended for use in the electromagnetic environment specified below. The customer or the user of the BC 57 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the BC 57, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Guidance and manufacture's declaration – electromagnetic immunity

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength

in the location in which the BC 57 is used exceeds the applicable RF compliance level above, the BC 57 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the BC 57.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the BC 57

The BC 57 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BC 57 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BC 57 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer North America is under license. Other trademarks and trade names are those of their respective owners. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.